



**Mt. Emey™**

# Toddler & Youth Sizing Chart


**IMPORTANT PRINTING INSTRUCTION:**  
 Make sure to set page scaling to none, or turn off "fit to page".  
 To check that your sizing chart printed accurately, the big rectangle sizing chart box should measure 4" wide x 10" high.

## Length Measurement

- 1 Place heel on chart**
  - Align heel of foot/AFO/KAFO with the bottom line.
  - If foot is longer than brace, include foot
- 2 Trace foot/AFO/KAFO**
  - Match longest point to number on right
  - If you are between or slightly above line, round up
- 3 Necessary Adjustments for Circumference**

For all AFOs, DAFO<sup>®</sup>s, KAFOs, SMOs with:

- If you are between or slightly above line, round up:  
**Increase 1/2 full size**
- Articulating hinges, posterior stop and instep strapping:  
**Increase 1 full size**
- Toe strapping, the additions above will change the fit of the shoe:  
**Increase 1/2 full sizes**

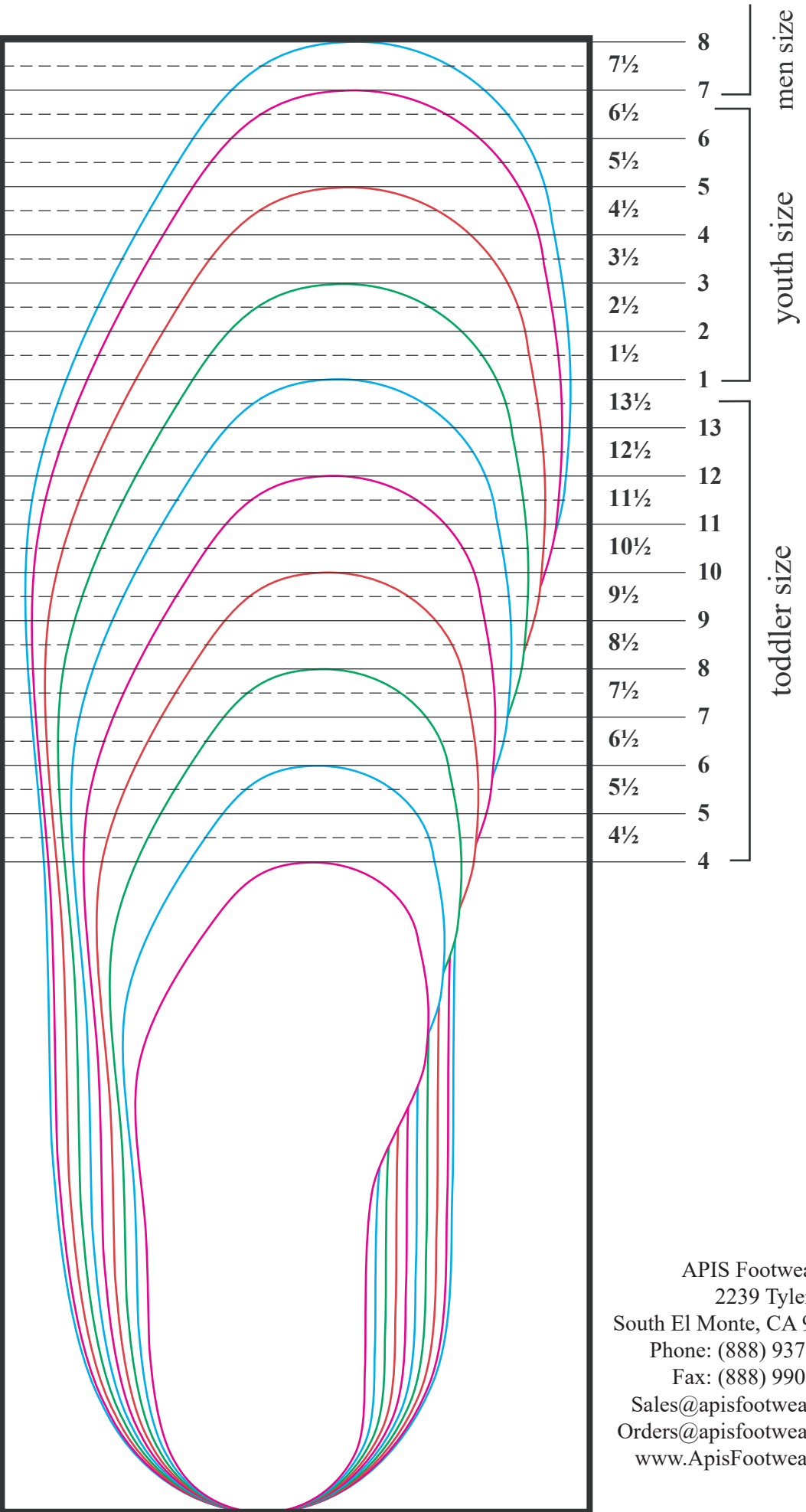

APIS Footwear Co.  
 2239 Tyler Ave.  
 South El Monte, CA 91733  
 Phone: (888) 937-2747  
 Fax: (888) 990-2245  
 Sales@apisfootwear.com  
 Orders@apisfootwear.com  
 www.ApisFootwear.com